


Care Lifts Parts List

- 1 – 400700 Scale Display
- 2 – 400940 Hand Control
- 3 – 400092 Padded Actuator Cover
- 4 – 400199 Lift Mast
400171 600 lb Lift Mast
- 5 – 400902 Receiver Box
- 6 – 400901 Battery Pack
- 7 – 400585 Spreader Handle
- 8 – 400535 Pivot Bar
- 9 – 400540 Linkage Rod
- 10 – 400325 Nylock Nut
- 11 – 400335 Lock Washer
- 12 – 400315 Rear Wheel
- 13 – 400150 Flat Washer
- 14 – 400575 Lock Washer
- 15 – 400550 Right Hand Ball Joint
400545 Left Hand Ball Joint
- 16 – 400560 Right Hand Nut
400555 Left Hand Nut
- 17 – 400499 Lift Leg
- 18 – 400599 Lift Base
400170 600 lb Lift Base
- 19 – 400510 Lift Base Cover
- 20 – 400410 Lift Leg Cover
- 21 – 400320 Front Wheel
400405 Cap Screw for Wheel
400325 Nylock Nut
- 22 – 400110 Mast Protector
- 23 – 400305 Lift Actuator
- 24 – 400137 Lift Actuator Bolt
- 25 – 400245 Nylon Washer for Actuator
- 26 – 400606 Hanger Bar
- 27 – 400096 Padded Hanger Bar Cover
- 28 – 400093 Padded Yoke Cover
- 29 – 400600 Scale Hanger Assembly
400607 Blank Scale Hanger
- 30 – 400625 Load Cell Housing
- 31 – 400620 Boom Head Cushion
- 32 – 400298 Lift Boom
- 33 – 400299 Boom Support
- 34 – 400135 Flat Washer
- 35 – 400125 Boom/Mast Bolt

- #400001 Care Lift
- #400019 Care Bâr Lift
- #400000 LIBI Lift
- #400003 Care Lift-n-Weigh
- #400013 Care Bâr Lift-n-Weigh

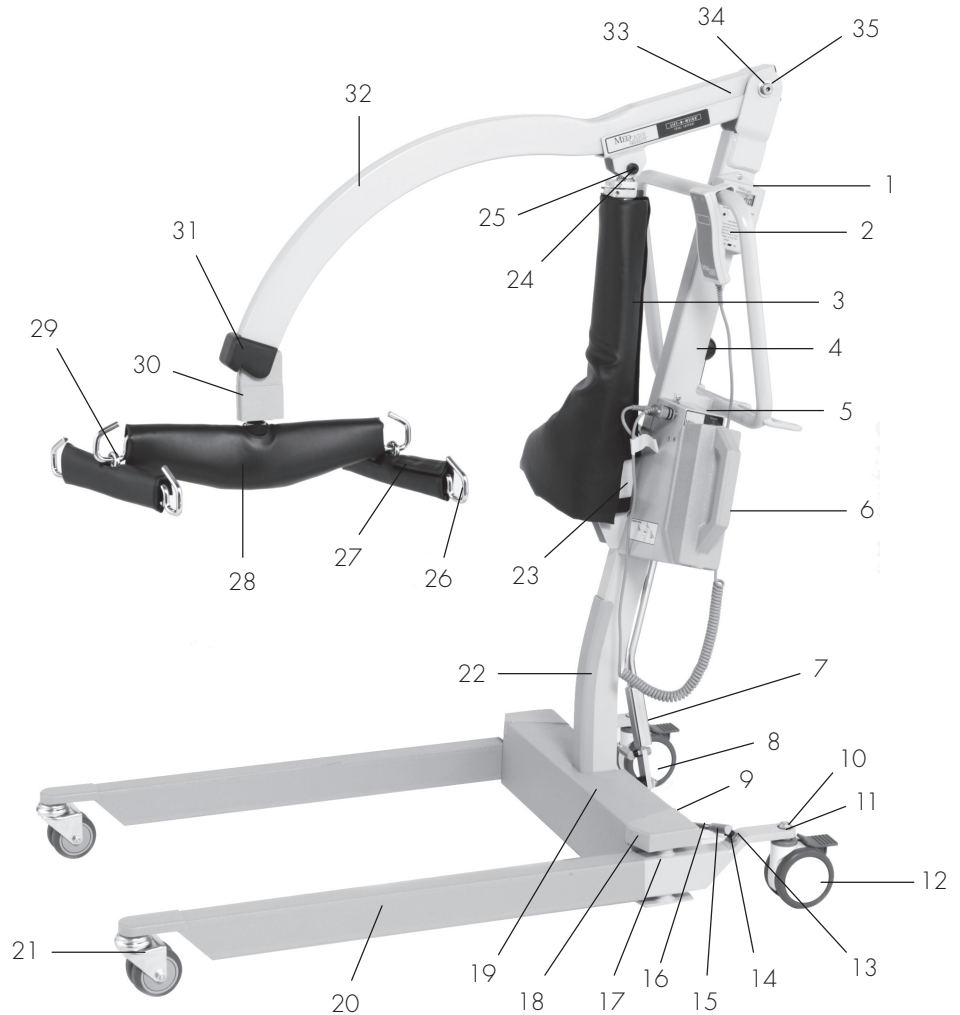

Items Not Pictured

- | | |
|-------------------------------------|-------------------------------------|
| 400800 Charger | 401520 Lock Washer |
| 400421 Leg Bolt | 400246 Nylon Boom Washer |
| 400210 T-shaped Boom Bushing | 400525 Spherical Washer |
| 400590 Ball for Handle | 400640 Nylock Nut for Scale Hanger |
| 400138 Nylon Nut for Actuator Bolts | 400150 Flat Mast Washer |
| 400565 Nylock Nut for Ball Joints | 401520 Lock Washer for Scale Hanger |
| 400520 Black Spring | 400760 Power Cable for Scale |
| 400325 Nylock Nut for Pivot Bar | 400150 Flat Washer for Scale Hanger |
| 400132 Grommet on Mast | 403206 Touch-up Paint Bottle |
| 400226 Scale Hanger Bolt | 401511 Rubber Padding in Base Cover |
| 400220 Mast Cap | |