


Care Stands Parts List

- 1 – 400700 Scale Display
- 2 – 400940 Hand Control
- 3 – 402199 SNW Mast
401199 Stand Mast
- 4 – 401946 Arm Cap Screw
- 5 – 400902 Receiver Box
- 6 – 400901 Battery Pack
- 7 – 400092 Padded Actuator Cover
400099 Runner Actuator Cover
- 8 – 401350 Foot Pedal
400325 Nylock Nut
400585 Leg Spreader Handle
- 9 – 400540 Linkage Rod
- 10 – 400325 Nylock Nut
- 11 – 400335 Lock Washer
- 12 – 400315 Rear Wheel
- 13 – 400150 Flat Washer
- 14 – 400575 Lock Washer
- 15 – 400550 Right Hand Ball Joint
400545 Left Hand Ball Joint
- 16 – 400560 Right Hand Nut
400555 Left Hand Nut
- 17 – 401499 Stand Leg
- 18 – 402599 SNW Base
401599 Stand Base
- 19 – 401515 SNW Base Cover
401510 Stand Base Cover
- 20 – 401410 Stand Leg Cover
- 21 – 400320 Front Wheel
400405 Cap Screw for Wheel
400325 Nylock Nut
- 22 – 402596A SNW Foot Plate
401596 Stand Foot Plate
- 23 – 401512 Foot Plate Cover
- 24 – 401306 Stand Actuator
401336 500 lb Runner Actuator
- 25 – 401009 Knee Pad Assembly
401010 Knee Pad
401012 Velcro Strap
401198 Knee Pad Bracket
401175 Plastic Glide
- 26 – 401275 Hand Grips
- 27 – 400097 Padded Arm Cover
- 28 – 401298 Stand Arm
- 29 – 400220 Mast Cap

- #400002 Care Stand
- #400005 Care Stand-n-Weigh®
- #400006 Care Bär Stand
- #400007 Care Bär Stand-n-Weigh®
- #400021 LIBI Stand

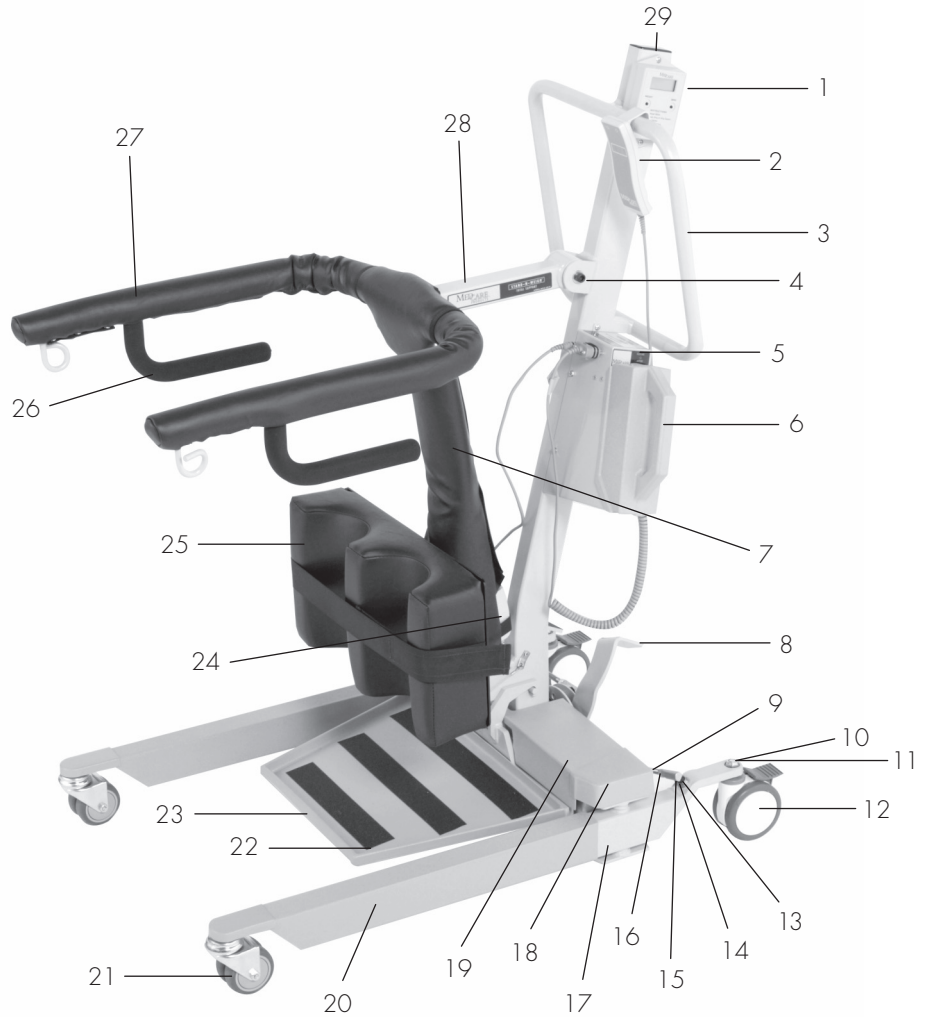

Items Not Pictured

- | | |
|--|-------------------------------------|
| 400800 Charger | 400575 Mast Lock Washer |
| 400421 Leg Bolt | 401521 Bolt Under Base |
| 402215 Base Plate Screw | 400150 Flat Mast Washer |
| 400247 Nylon Washer for Arm | 401520 Lock Washer Under Base |
| 402596B 1/4" L Shaped Plate | 400120 Small Phillips Screw |
| 400245 Nylon Washer for Actuator Bolts | 400235 Flat Washer Under Base |
| 401706 Black Plug | 401511 Rubber Padding in Base Cover |
| 400138 Nylock Nut for Actuator Bolt | 401301 Stand Arm Pivot Pin |
| 401699 SNW Load Cell | 402001 Square Base Plate |
| 400132 Grommet on Mast | 400760 Power Cable for Scale |
| 401570 Pin for Safety Cable | 401137 Stand Actuator Bolt |
| | 401134 Special Stand Arm Washer |
| | 403206 Touch-up Paint Bottle |