


# Car Extractor Parts List

#450000 Car Extractor

- 1 – 400990 Hand Control
- 2 – 400092 Padded Actuator Cover
- 3 – 450198 Mast
- 4 – 400902 Receiver Box
- 5 – 400901 Battery Pack
- 6 – 401350 Foot Pedal
- 7 – 400540 Linkage Rod
- 8 – 400325 Nylock Nut
- 9 – 400335 Lock Washer
- 10 – 400560 Right Hand Nut
- 400555 Left Hand Nut
- 11 – 400150 Flat Washer
- 12 – 400575 Lock Washer
- 13 – 400550 Right Hand Ball Joint
- 400545 Left Hand Ball Joint
- 14 – 400315 Rear Wheel
- 15 – 400499 Lift Leg
- 16 – 400599 Lift Base
- 17 – 400510 Lift Base Cover
- 18 – 400410 Lift Leg Cover
- 19 – 400320 Front Wheel
- 400405 Cap Screw for Wheel
- 400325 Nylock Nut
- 20 – 400110 Mast Protector
- 21 – 400305 Lift Actuator
- 22 – 400137 Lift Actuator Bolt
- 23 – 400245 Nylon Washer for Actuator
- 24 – 400093 Padded Yoke Cover
- 25 – 400299 Boom Support
- 26 – 400620 Boom Head Cushion
- 27 – 400296 Extractor Boom
- 28 – 400135 Flat Washer
- 29 – 400125 Boom/Mast Bolt

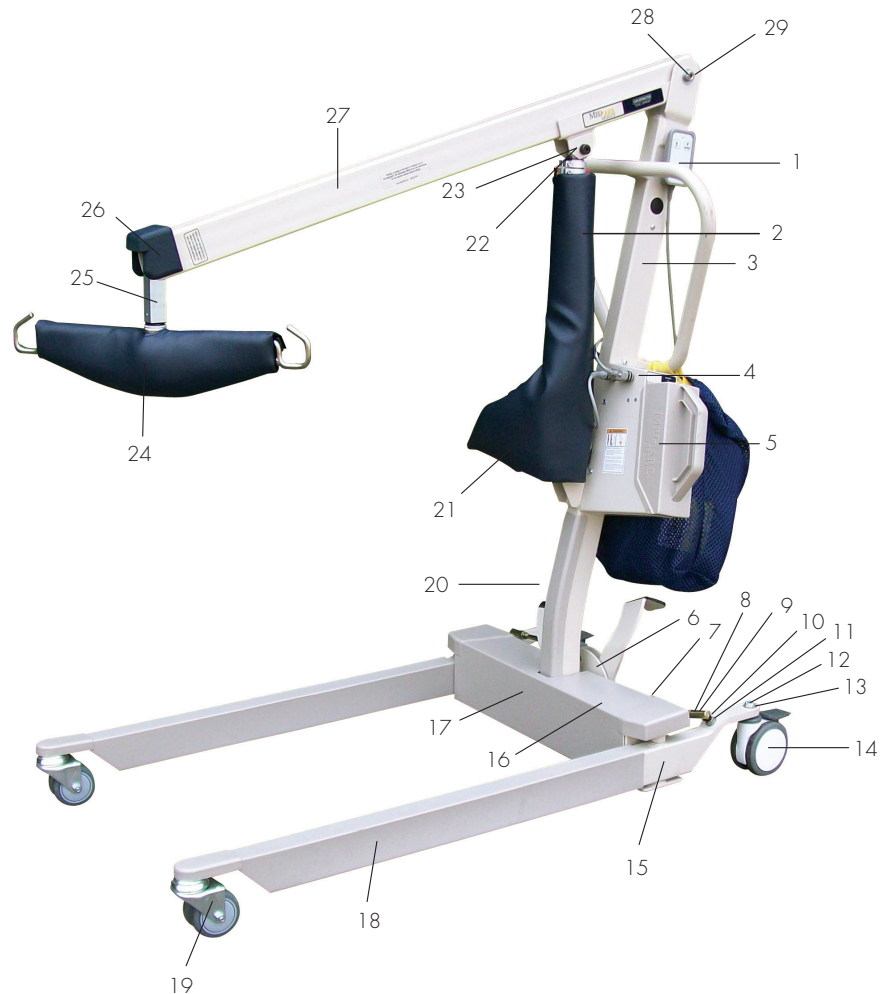

## Items Not Pictured

- |                                     |                                     |
|-------------------------------------|-------------------------------------|
| 400800 Charger                      | 400585 Spreader Handle              |
| 400421 Leg Bolt                     | 400535 Pivot Bar                    |
| 400210 T-shaped Boom Bushing        | 401520 Lock Washer                  |
| 400590 Ball for Handle              | 400246 Nylon Boom Washer            |
| 400138 Nylon Nut for Actuator Bolts | 400525 Spherical Washer             |
| 400565 Nylock Nut for Ball Joints   | 400640 Nylock Nut for Scale Hanger  |
| 400520 Black Spring                 | 400150 Flat Mast Washer             |
| 400325 Nylock Nut for Pivot Bar     | 401520 Lock Washer for Scale Hanger |
| 400132 Grommet on Mast              | 400150 Flat Washer for Scale Hanger |
| 400226 Scale Hanger Bolt            | 403206 Touch-up Paint Bottle        |
| 400220 Mast Cap                     | 401511 Rubber Padding in Base Cover |