


# Care Lifts Parts List

- 1 – 400700 Scale Display
- 2 – 400990 Hand Control
- 3 – 400092 Padded Actuator Cover
- 4 – 400198 Lift Mast
- 400177 600 lb Lift Mast
- 5 – 400902 Receiver Box
- 6 – 400901 Battery Pack
- 8 – 401350 Foot Pedal
- 9 – 400540 Linkage Rod
- 10 – 400325 Nylock Nut
- 11 – 400335 Lock Washer
- 12 – 400315 Rear Wheel
- 13 – 400150 Flat Washer
- 14 – 400575 Lock Washer
- 15 – 400550 Right Hand Ball Joint
- 400545 Left Hand Ball Joint
- 16 – 400560 Right Hand Nut
- 400555 Left Hand Nut
- 17 – 400499 Lift Leg
- 18 – 400599 Lift Base
- 400170 600 lb Lift Base
- 19 – 400510 Lift Base Cover
- 20 – 400410 Lift Leg Cover
- 21 – 400320 Front Wheel
- 400405 Cap Screw for Wheel
- 400325 Nylock Nut
- 22 – 400110 Mast Protector
- 23 – 400305 Lift Actuator
- 24 – 400137 Lift Actuator Bolt
- 25 – 400245 Nylon Washer for Actuator
- 26 – 400606 Hanger Bar
- 27 – 400096 Padded Hanger Bar Cover
- 28 – 400093 Padded Yoke Cover
- 29 – 400600 Scale Hanger Assembly
- 400607 Blank Scale Hanger
- 30 – 400625 Load Cell Housing
- 31 – 400620 Boom Head Cushion
- 32 – 400298 Lift Boom
- 33 – 400299 Boom Support
- 400172 600 lb Boom Support
- 34 – 400135 Flat Washer
- 35 – 400125 Boom/Mast Bolt

- #400001 Care Lift**
- #400019 Care Bâr Lift**
- #400000 LIBI Lift**
- #400003 Care Lift-n-Weigh**
- #400013 Care Bâr Lift-n-Weigh**

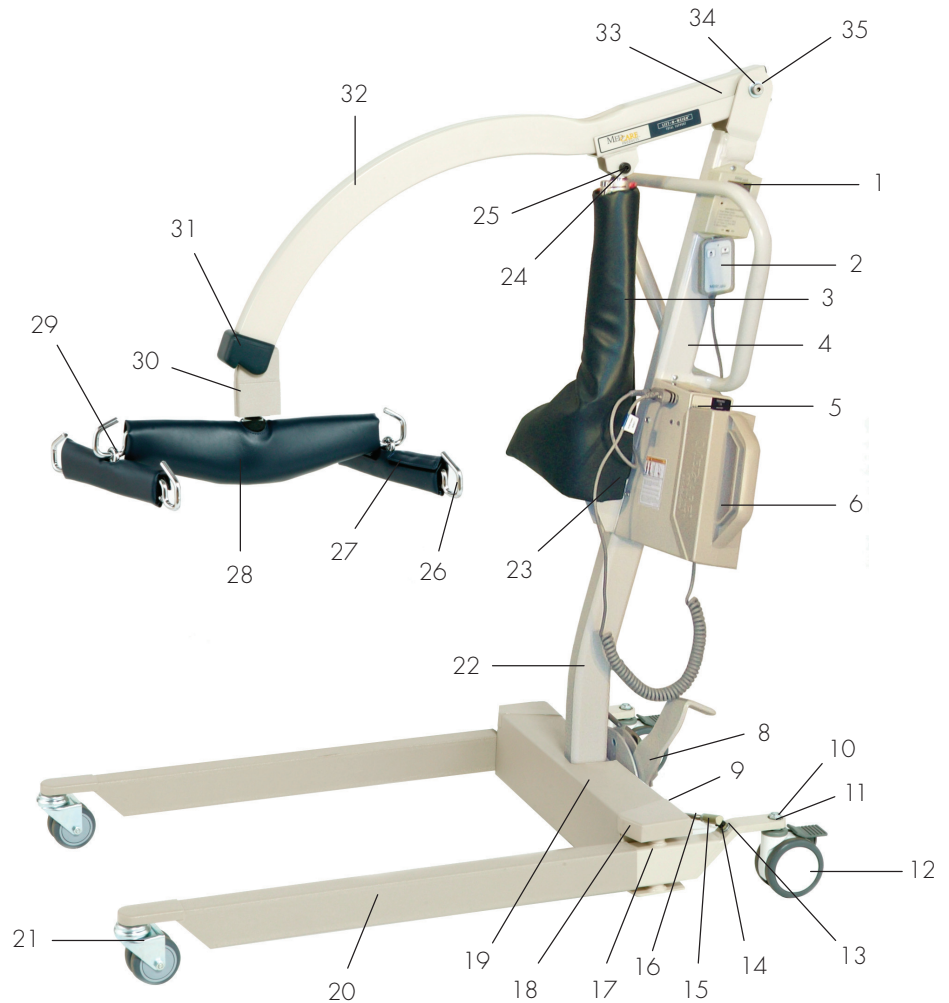

## Items Not Pictured

- 400800 Charger
- 400421 Leg Bolt
- 400210 T-shaped Boom Bushing
- 400590 Ball for Handle
- 400138 Nylon Nut for Actuator Bolts
- 400565 Nylock Nut for Ball Joints
- 400520 Black Spring
- 400325 Nylock Nut for Pivot Bar
- 400132 Grommet on Mast
- 400226 Scale Hanger Bolt
- 400220 Mast Cap
- 400585 Spreader Handle
- 400535 Pivot Bar
- 401520 Lock Washer
- 400246 Nylon Boom Washer
- 400525 Spherical Washer
- 400640 Nylon Nut for Scale Hanger
- 400150 Flat Mast Washer
- 401520 Lock Washer for Scale Hanger
- 400760 Power Cable for Scale
- 400150 Flat Washer for Scale Hanger
- 403206 Touch-up Paint Bottle
- 401511 Rubber Padding in Base Cover