

Medcare's 450/625 Ceiling Lift Cleaning in the Operating Room

Medcare's 450/625 Ceiling Lift Strap Cleaning and Disinfection:

The use of Ready-to-Use Accelerated Hydrogen Peroxide (AHP) with 0.5% hydrogen peroxide, found in AccelTB/OxivirTB, has been approved and tested, for the cleaning and disinfecting of the Medcare 450/625 Ceiling Lift Strap. Other disinfectants with equivalent makeup can be used.

Additional Medcare 450/625 Ceiling Lift Cleaning and Disinfecting Information:

With a damp dust cloth the following areas should be cleaned and/or disinfected during the terminal cleaning process:

- **Track (Inside and Outside):** Damp dust with AHP, Isopropyl Alcohol or Quaternary Ammonium
- **Carry Bar including padding:** Damp dust with Isopropyl Alcohol
- **Hand Control including cord:** Damp dust with Isopropyl Alcohol
- **Motor Cover:** Damp dust with Isopropyl Alcohol
- **Emergency Lowering Cord:** Damp dust with Isopropyl Alcohol

Caution: Great care should be taken to ensure that no liquids get inside the lift. The lift is not drip or water proof. Failure to protect the lift from liquids may result in damage to the lift and/or may cause injury.

Medcare Products takes every step to ensure a safe working environment, and every hospital has different standards for Infection Control. This is meant to be a guideline for approved and recommended cleaners/solvents and areas of the equipment that should be cleaned.