



Medcare's Lift, Stand & Car Extractor Cleaning

The following technical bulletin has been issued due to the number of recent inquiries regarding the cleaning and disinfection of the Medcare floor machines.

Wipe all metal, plastic and padding down with a damp cloth using warm water and mild detergent. When higher levels of disinfection are required; use Accelerated Hydrogen Peroxide (AHP) with 0.5% hydrogen peroxide which is found in Accel TB or Oxivir TB or 70% Isopropyl Alcohol. These have been approved and tested on Medcare floor machines. Other disinfectants with equivalent chemical makeup can be used.

NOTE: Do not use petroleum based solvents, trichloroethylene or similar products in cleaning the surface Medcare's floor machines, as they may damage the plastic. If a hot air dryer is used, the temperature must not exceed 175° F.