

# Transfer & Position Aids Cleaning Recommendations

## MedSheet & MedSheet Tube

- Machine wash in soap solution
- Maximum washing temperature is 140 F / 60 C
- DO NOT USE BLEACH
- Rinse thoroughly
- Hang Dry
- DO NOT PLACE IN DIRECT CONTACT WITH HEAT SOURCE
- DO NOT DRY CLEAN



Medsheet Set



Medsheet Tube


## Gait Belts

- Machine wash in soap solution
- Maximum washing temperature is 140 F / 60 C
- Maximum drying temperature is 104 F / 40 C
- DO NOT USE BLEACH



## One Way Slide

- Machine wash in soap solution
- Maximum washing temperature is 140 F / 60 C
- Maximum drying temperature 140 F / 60 C
- DO NOT USE FABRIC CONDITIONER & BLEACH WHICH WILL DESTROY THE GLIDING EFFECT
- DO NOT PLACE IN DIRECT CONTACT WITH HEAT SOURCE
- DO NOT DRY CLEAN

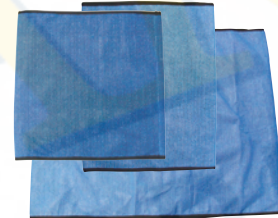

## Q2 Board


- Wash in soap solution
- Wipe down with 0.5% Accelerated Hydrogen Peroxide
- Wipe down with alcohol wipes



## Lateral Assist

### Cover:

- Machine wash in soap solution
- Maximum washing temperature is 140 F / 60 C
- DO NOT USE BLEACH
- Rinse thoroughly
- Hang Dry
- DO NOT PLACE IN DIRECT CONTACT WITH HEAT SOURCE
- DO NOT DRY CLEAN

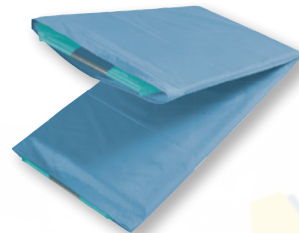

### Board:

- Wash in soap solution
- Wipe down with 0.5% Accelerated Hydrogen Peroxide
- Wipe down with alcohol wipes without chlorine